



**REF 10100**  
**ANA Screen**



# Instruction Manual

## Table of Contents

---

1	Intended Use .....	2
2	Clinical Application and Principle of the Assay .....	2
3	Kit Contents .....	3
4	Storage and Shelf Life .....	4
5	Precautions of Use.....	4
6	Sample Collection, Handling and Storage.....	5
7	Assay Procedure .....	6
8	Qualitative Interpretation .....	9
9	Technical Data.....	10
10	Performance Data.....	10
11	Disposal.....	11
12	Literature .....	11



AIDA GmbH  
Dr.-Karl-Aschoff-Straße 9  
55543 Bad Kreuznach  
Germany  
Phone: +49 671 92065090  
Fax: +49 671 92065091  
Website: [www.aida-diagnostics.com](http://www.aida-diagnostics.com)  
Mail: [info@aida-diagnostics.com](mailto:info@aida-diagnostics.com)



	Product Ref.	10100
	Product Desc.	ANA Screen
	Manual Rev. No.	006: 2024-02-12

## 1 Intended Use

**ANA Screen** is a solid phase enzyme immunoassay for the combined qualitative detection of IgG antibodies against eight cellular and nuclear antigens in human serum. Each well is coated with recombinant 70 kDa U1-snRNP, SS-B, SS-A 52 kDa, Scl 70, centromere protein B (CenpB), Jo-1 and highly purified native human snRNP/Sm, Sm and SS-A 60 kDa.

The assay is a tool in the differential diagnosis of systemic rheumatic diseases.

## 2 Clinical Application and Principle of the Assay

Anti-nuclear antibodies (ANA) are an important tool for the differential diagnosis of systemic rheumatic diseases. Indirect immunofluorescence test (IFT) on eucaryotic cells like HeLa has been the established method for the detection of ANAs. Single antibody specificities are distinguished by fluorescence patterns but more specific testing by ELISAs employing the target antigens are available too for a simple and reliable differentiation of ANAs.

ANAs are especially found in active and inactive systemic lupus erythematosus (SLE), mixed connective tissue diseases (MCTD), scleroderma, Sjögren`s syndrome, polymyositis.

### Antibodies against:

- Sm (Smith antigen) are directed against core proteins (B,B`, D1-D3, E, F, G) of small nuclear ribonucleoproteins (snRNPs). Anti-Sm as well as antibodies against double stranded DNA (dsDNA) are highly specific for SLE and thus are included in diagnostic and classification criteria for SLE.
- U1 snRNP is directed to the 70 kDa protein of U1 snRNP. They are pathognomic for MCTD but do also occur in SLE. A high titer of antibodies against this antigen is typical for the Sharp-Syndrome.
- snRNP/Sm are directed against Sm and U1-snRNP proteins (70 kDa, A and C). They occur in SLE, Sjögren`s syndrome, scleroderma and polymyositis.
- SS-A (Ro; soluble cytoplasmic and/or nuclear ribonucleoproteins of 52 kDa and 60 kDa) and antibodies against SS-B (La; 48 kDa protein associated with RNA polymerase III) are mainly found in high titers for primary and secondary Sjögren`s syndrome but also in SLE, congenital heartblock and neonatal lupus.
- Scl-70 are directed against DNA-topoisomerase I. They are highly specific for systemic scleroderma and give a hint for a severe course.
- CenpB (80kDa centromere protein B) are typical for the CREST-Syndrome (69% of CREST-patients), which is a more protracted type of systemic sclerosis
- Jo-1 are directed against histidyl-tRNA synthetase (cytoplasmic protein involved in protein biosynthesis) and are found in 20-40 % of patients with polymyositis and dermatomyositis.

	Product Ref.	10100
	Product Desc.	ANA Screen
	Manual Rev. No.	006: 2024-02-12

### Principle of the test

Serum samples diluted 1:101 are incubated in the microplates coated with the specific antigen. Patient's antibodies, if present in the specimen, bind to the antigen. The unbound fraction is washed off in the following step. Afterwards anti-human immunoglobulins conjugated to horseradish peroxidase (conjugate) are incubated and react with the antigen-antibody complex of the samples in the microplates. Unbound conjugate is washed off in the following step. Addition of TMB-substrate generates an enzymatic colorimetric (blue) reaction, which is stopped by diluted acid (color changes to yellow). The intensity of color formation from the chromogen is a function of the amount of conjugate bound to the antigen-antibody complex and this is proportional to the initial concentration of the respective antibodies in the patient sample.

## 3 Kit Contents

TO BE RECONSTITUTED				
Item	Quantity	Cap color	Solution color	Description / Contents
Sample Buffer (5x)	1 x 20ml	White	Yellow	5 x concentrated Tris, sodium chloride (NaCl), , sodium azide < 0.1% (preservative)
Wash Buffer (50x)	1 X 20ml	White	Green	50 x concentrated Tris, NaCl, Tween 20, sodium azide < 0.1% (preservative)
READY TO USE				
Item	Quantity	Cap color	Solution color	Description / Contents
Negative Control	1 x 1.5ml	Green	Colorless	Control material (diluted), bovine serum albumin (BSA), sodium azide < 0.1% (preservative)
Positive Control	1 x 1.5ml	Red	Yellow	Control material (diluted), bovine serum albumin (BSA), sodium azide < 0.1% (preservative)
Cut-off Calibrator	1 x 1.5ml	Blue	Yellow	Calibrator material (diluted), bovine serum albumin (BSA), sodium azide < 0.1% (preservative)
Conjugate, IgG	1 x 15ml	Blue	Blue	Containing: Immunoglobulins conjugated to horseradish peroxidase, bovine serum albumin (BSA),
TMB Substrate	1 x 15ml	Black	Colorless	Stabilized tetramethylbenzidine and hydrogen peroxide (TMB/H <sub>2</sub> O <sub>2</sub> )
Stop Solution	1 x 15ml	White	Colorless	1M Hydrochloric Acid
Microtiter plate	12 x 8 well strips	N/A	N/A	With breakaway microwells. Refer to paragraph 1 for coating.
* Color increasing with concentration				
MATERIALS REQUIRED, BUT NOT PROVIDED				
Microtiter plate reader 450 nm reading filter and recommended 620 nm reference filter (600-690 nm). Glass ware (cylinder 100-1000ml), test tubes for dilutions. Vortex mixer, precision pipettes (10, 100, 200, 500, 1000 µl) or adjustable multipipette (100-1000µl). Microplate washing device (300 µl repeating or multichannel pipette or automated system), adsorbent paper. Our tests are designed to be used with purified water according to the definition of the United States Pharmacopeia (USP 26 - NF 21) and the European Pharmacopeia (Eur.Ph. 4th ed.).				

	Product Ref.	10100
	Product Desc.	ANA Screen
	Manual Rev. No.	006: 2024-02-12

## 4 Storage and Shelf Life

Store all reagents and the microplate at 2-8°C/35.6-46.4°F, in their original containers. Once prepared, reconstituted solutions are stable at 2-8°C/35.6-46.4°F for 1 month. Reagents and the microplate shall be used within the expiry date indicated on each component, only. Avoid intense exposure of TMB solution to light. Store microplates in designated foil, including the desiccant, and seal tightly.

## 5 Precautions of Use

### 5.1 Health hazard data

**THIS PRODUCT IS FOR IN VITRO DIAGNOSTIC USE ONLY.** Thus, only staff trained and specially advised in methods of in vitro diagnostics may perform the kit. Although this product is not considered particularly toxic or dangerous in conditions of the intended use, refer to the following for maximum safety:

#### ***Recommendations and precautions***

This kit contains potentially hazardous components. Though kit reagents are not classified being irritant to eyes and skin we recommend to avoid contact with eyes and skin and wear disposable gloves.

**WARNING !** Calibrators, Controls and Buffers contain sodium azide ( $\text{NaN}_3$ ) as a preservative.  $\text{NaN}_3$  may be toxic if ingested or adsorbed by skin or eyes.  $\text{NaN}_3$  may react with lead and copper plumbing to form highly explosive metal azides. On disposal, flush with a large volume of water to prevent azide build-up. Please refer to decontamination procedures as outlined by CDC or other local/national guidelines.

**Do not smoke, eat or drink when manipulating the kit. Do not pipette by mouth.**

All biological source material used for some reagents of this kit has been tested by approved methods and found negative for HbsAg, Hepatitis C and HIV 1. However, no test can guarantee the absence of viral agents in such material completely. Thus handle these as if capable of transmitting infectious diseases and according to national requirements.

The kit contains material of animal origin as stated in the table of contents, handle according to national requirements.

	Product Ref.	10100
	Product Desc.	ANA Screen
	Manual Rev. No.	006: 2024-02-12

## 5.2 General directions for use

In case that the product information, including the labeling, is defective or incorrect please contact the manufacturer or the supplier of the test kit.

Do not mix or substitute Controls, Calibrators, Conjugates or microplates from different lot numbers. This may lead to variations in the results.

Allow all components to reach room temperature (20-32°C/68-89.6°F) before use, mix well and follow the recommended incubation scheme for an optimum performance of the test.

**Incubation: We recommend test performance at 30°C/86°F for automated systems.**

Never expose components to higher temperature than 37°C/ 98.6°F.

Always pipette substrate solution with brand new tips only. Protect this reagent from light. Never pipette conjugate with tips used with other reagents prior.

**A definite clinical diagnosis should not be based on the results of the performed test only, but should be made by the physician after all clinical and laboratory findings have been evaluated. The diagnosis is to be verified using different diagnostic methods.**

## 6 Sample Collection, Handling and Storage

Use preferentially freshly collected serum samples. Blood withdrawal must follow national requirements. Do not use icteric, lipemic, hemolysed or bacterially contaminated samples. Sera with particles should be cleared by low speed centrifugation (<1000 x g). Blood samples should be collected in clean, dry and empty tubes.

After separation, the serum samples should be used during the first 8h, respectively stored tightly closed at 2-8°C/35.6-46.4°F up to 48h, or frozen at -20°C/-4°F for longer periods. (Thomas: Labor und Diagnose; CLSI Guideline GP 44-A4)

	Product Ref.	10100
	Product Desc.	ANA Screen
	Manual Rev. No.	006: 2024-02-12

## 7 Assay Procedure

### 7.1 Preparations prior to starting

Dilute concentrated reagents:

Dilute the concentrated sample buffer 1:5 with distilled water (e.g. 20 ml plus 80 ml).

Dilute the concentrated wash buffer 1:50 with distilled water (e.g. 20 ml plus 980 ml).

To avoid mistakes we suggest to mark the cap of the different calibrators.

#### **Samples:**

Dilute serum samples 1:101 with sample buffer (1x)

e.g. 1000 µl sample buffer (1x) + 10 µl serum. Mix well !

#### **Washing:**

Prepare 20 ml of diluted wash buffer (1x) per 8 wells or 200 ml for 96 wells

e.g. 4 ml concentrate plus 196 ml distilled water.

#### **Automated washing:**

Consider excess volumes required for setting up the instrument and dead volume of robot pipette.

#### **Manual washing:**

Discard liquid from wells by inverting the plate. Knock the microwell frame with wells downside vigorously on clean adsorbent paper. Pipette 300 µl of diluted wash buffer into each well, wait for 20 seconds. Repeat the whole procedure twice again.

#### **Microplates:**

Calculate the number of wells required for the test. Remove unused wells from the frame, replace and store in the provided plastic bag, together with desiccant, seal tightly (2-8°C/35.6-46.4°F).



## 7.2 Pipetting Scheme

We suggest pipetting calibrators, controls and samples as follows:

For *QUALITATIVE* interpretation

	1	2	3	4...
A	NC	P2		
B	NC	P2		
C	CC	P3		
D	CC	P3		
E	PC	...		
F	PC	...		
G	P1	...		
H	P1	...		

PC: positive control

P1: patient 1


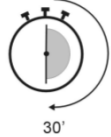
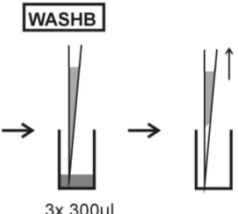
NC: negative control


P2: patient 2


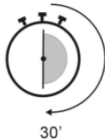
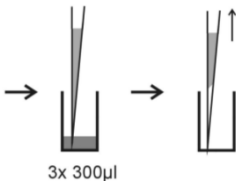


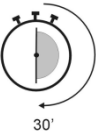



CC: cut-off calibrator

P3: patient 3

## 7.3 Test Steps

Step	Description
1.	Ensure preparations from step 7.1 above have been carried out prior to pipetting.
2.	Use the following steps in accordance with qualitative interpretation results desired:
<b>CONTROLS &amp; SAMPLES</b>	
3.	 <p>Pipette into the designated wells as described in chapter 7.2 above, 100 µl of either: Cut-off Calibrator (CC) for <i>QUALITATIVE</i> interp. and 100 µl of each of the following:</p> <ul style="list-style-type: none"> <li>Negative control (NC) and Positive control (PC), and</li> <li>Patients diluted serum (P1, P2...)</li> </ul>
4.	 <p>Incubate for 30 minutes at 20-32°C/68-89.6°F.</p>
5.	 <p>Wash 3x with 300 µl washing buffer (diluted 1:50).</p>

<div> autoimmune diagnostic assays</div>		Product Ref.	10100
		Product Desc.	ANA Screen
		Manual Rev. No.	006: 2024-02-12

CONJUGATE	
6.	<div><div>CONJ</div><div> +100 µl</div></div> <div>Pipette 100 µl conjugate into each well.</div>
7.	<div> 30'</div> <div>Incubate for 30 minutes at 20-32°C/68-89.6°F.</div>
8.	<div><div>WASHB</div><div> 3x 300µl</div></div> <div>Wash 3x with 300 µl washing buffer (diluted 1:50).</div>
SUBSTRATE	
9.	<div><div>SUB</div><div> +100 µl</div></div> <div>Pipette 100 µl TMB substrate into each well.</div>
10.	<div>  30'</div> <div>Incubate for 30 minutes at 20-32°C/68-89.6°F, protected from intense light.</div>
STOP	
11.	<div><div>STOP</div><div> +100 µl</div></div> <div>Pipette 100 µl stop solution into each well, using the same order as pipetting the substrate.</div>
12.	<div> 5'</div> <div>Incubate 5 minutes minimum.</div>
13.	Agitate plate carefully for 5 sec.
14.	<div><div>OD<sub>450</sub> - OD<sub>620</sub> 450/620 nm</div><div>Read absorbance at 450 nm (recommended 450/620 nm) within 30 minutes.</div></div>

## 8 Qualitative Interpretation

Read the optical density of the cut-off calibrator and the patient samples. Compare patient ODs with the OD of the cut-off calibrator. For **qualitative interpretation** we recommend to consider sera within a range of 20% around the cut-off value as equivocal. All samples with higher ODs are considered positive, samples with lower ODs are considered negative.

**Negative:** OD patient < 0.8 x OD cut-off  
**Equivocal:** 0.8 x OD cut-off ≤ OD patient ≤ 1.2 x OD cut-off  
**Positive:** OD patient > 1.2 x OD cut-off

Calibrators	O.D. 450/620 nm	CV % (Variation)
Negative Control	0.047	2.6
Cut-off Calibrator	0.350	1.8
Positive Control	1.259	0.7

**Example of interpretation.**

**We recommend pipetting cut-off calibrator in parallel for each run.**

Cut-off calibrator	Patient sample	OD Quotient	Interpretation
0.35 OD	0.25 OD	0.75	Negative
0.35 OD	0.40 OD	1.14	Equivocal
0.35 OD	0.56 OD	1.60	Positive
0.35 OD	1.75 OD	5.00	Positive

**Do not use this example for interpreting patients results!**

We recommend to retest samples, that are borderline. For lot specific data, see enclosed quality control leaflet. Medical laboratories might perform an in-house Quality Control by using own controls and/or internal pooled sera, as foreseen by national regulations.

Each laboratory should establish its own normal range based upon its own techniques, controls, equipment and patient population according to their own established procedures.

In case that the values of the controls do not meet the criteria the test is invalid and has to be repeated.

The following technical issues should be verified: Expiration dates of (prepared) reagents, storage conditions, pipettes, devices, photometer, incubation conditions and washing methods.

If the items tested show aberrant values or any kind of deviation or that the validation criteria are not met without explicable cause please contact the manufacturer or the supplier of the test kit.

For **semi-quantification** of the results, each patient-OD value can be expressed by the Index-Value. The Index-Value is calculated by dividing the patient-OD by the cut-off OD:

	Product Ref.	10100
	Product Desc.	ANA Screen
	Manual Rev. No.	006: 2024-02-12

$$\text{Index Value} = \frac{\text{OD (patient sample)}}{\text{OD (cut-off calibrator)}}$$

Negative:	Index Value	< 0.8
Equivocal:	0.8 ≤ Index Value	≤ 1.2
Positive:	Index Value	> 1.2

## 9 Technical Data

Sample material:	serum
Sample volume:	10 µl of sample diluted 1:101 with 1x sample buffer
Total incubation time:	90 minutes at 20-32°C/68-89.6°F
Storage:	at 2-8°C/35.6-46.4°F use original vials only.
Number of determinations:	96 tests

## 10 Performance Data

### 10.1 Normal Range

Sera of healthy donors have been investigated on **ANA Screen** and resulted in the following distribution:

Number of Samples	negative	borderline	positive
80	73 (91.3 %)	0 (0 %)	7 (8.8 %)

We also recommend that each laboratory should establish its own normal range.

### 10.2 Precision

Precision of test results obtained with **ANA Screen**, REF 10100 were assessed by the determination of the intra- and inter assay precision as well as the lot-to-lot variance by the analysis of multiple samples of different antibody activities.

Sample ID	Intra Assay Precision		Inter Assay Precision		LOT to LOT Precision	
	Mean (Index Value)	CV	Mean (Index Value)	CV	Mean (Index Value)	CV
Sample 1	0.76	8.3%	0.76	8.8%	0.78	4.8%
Sample 2	1.16	9.6%	1.16	13.8%	1.17	4.4%
Sample 3	2.34	4.6%	2.34	4.8%	2.30	5.4%
Sample 4	4.20	2.7%	4.20	3.1%	4.22	3.6%
Sample 5	6.05	1.1%	6.05	2.3%	6.30	3.5%

	Product Ref.	<b>10100</b>
	Product Desc.	ANA Screen
	Manual Rev. No.	006: 2024-02-12

## 10.3 Sensitivity and Specificity

### Analytical sensitivity

The analytical sensitivity has been assessed by multiple analysis of sample buffer and low positive samples and calculating the limit of detection.

For **ANA Screen**, REF 10100 a **LoD of 0.1 (Index value)** has been determined.

## 10.4 Calibration

The **ANA Screen** is calibrated against reference sera from the CDC (Centers for Disease Control and Prevention) Atlanta.

## 11 Disposal

Please observe the relevant statutory requirements!

## 12 Literature

**Peter JB, Shoenfeld Y (1996).** Autoantibodies. Elsevier Sciences B.V., Amsterdam.

**Froelich CH, Wallmann H, Skosey JL and Teodorescu M (1990).** Clinical value of an integrated ELISA system for the detection of 6 autoantibodies. The Journal of Rheumatology 17 (2): 192-200.




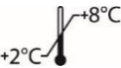

**Mierau R, Genth E (1998)** Autoantikörper bei systemischem Lupus erythematoses und verwandten Erkrankungen In: Thomas L. (Hrsg.) Labor und Diagnose TH-Books, Frankfurt, 15. Auflage: 843-851.

**Schmolke M, Oppermann M, Helmke K, Guder WG.** Antibody determination against ENA- a challenge for the routine laboratory Poster P59, 5 th Dresden Symposium on Autoantibodies, 2000.

**Tan EM, (1989)** Antinuclear antibodies: diagnostic markers for autoimmune diseases and probes for cell biology. Adv. Immunol 44: 93-151.

**Lothar Thomas:** Labor und Diagnose. Indikation und Bewertung von Laborbefunden für die medizinische Diagnostik., 8. Auflage, TH Books

**CLSI Guideline GP44-A4:** Procedures for the Handling and Processing of Blood Specimens for Common Laboratory Tests

<b>IVD</b>	- Diagnosi in vitro - Pour diagnostic in vitro - In Vitro Diagnostikum - Para uso Diagnóstico in vitro	- For in vitro diagnostic use - Para uso diagnóstico in vitro - In Vitro Διαγνωστικό μέσο
<b>REF</b>	- Numero d'ordine - Référence Catalogue - Bestellnummer - Número de catálogo	- Catalogue number - Numéro de catálogo - Αριθμός παραγγελίας
<b>LOT</b>	- Descrizione lotto - Lot - Chargen Bezeichnung - Lote	- Lot - Lote - Χαρακτηρισμός παρτίδας
<b>UDI</b>	- Identificatore univoco del dispositivo - Identifiant unique de l'appareil - eindeutige Produktidentifizierung - Identificador único do dispositivo	- Unique Device Identifier - Identificador único del dispositivo - Μοναδικό αναγνωριστικό συσκευής
<b>CE</b>	- Conformità europea - Déclaration CE de Conformité - Europäische Konformität - Declaração CE de Conformidade	- EC Declaration of Conformity - Declaración CE de Conformidad - Ευρωπαϊκή συμφωνία
	- 96 determinazioni - 96 tests - 96 Bestimmungen - 96 Testes	- 96 tests - 96 pruebas - 96 προσδιορισμοί
	- Rispettare le istruzioni per l'uso - Voir les instructions d'utilisation - Gebrauchsanweisung beachten - Ver as instruções de uso	- See instructions for use - Ver las instrucciones de uso - Λάβετε υπόψη τις οδηγίες χρήσης
	- Da utilizzarsi entro - Utilise avant le - Verwendbar bis - Utilizar antes de	- Use by - Utilizar antes de - Χρήση μέχρι
	- Conservare a 2-8°C - Conserver à 2-8°C - Lagerung bei 2-8°C - Conservar entre 2-8°C	- Store at 2-8°C (35.6-46.4°F) - Conservar a 2-8°C - Φυλάσσεται στους 2-8°C
	- Prodotto da - Fabriqué par - Hergestellt von - Fabricado por	- Manufactured by - Fabricado por - Κατασκευάζεται από
<b>CON</b> <b>+</b>	- Controllo positivo - Contrôle Positif - Positiv Kontrolle - Controllo positivo	- Positive Control - Control Positivo - Θετικός ορός ελέγχου
<b>CON</b> <b>-</b>	- Controllo negativo - Contrôle Négatif - Negativ Kontrolle - Controllo negativo	- Negative Control - Control Negativo - Αρνητικός ορός ελέγχου
<b>CO-CAL</b>	- Calibratore cut-off - Etalon Seuil - Grenzwert Kalibrator - Calibrador de cut-off	- Cut off Calibrator - Calibrador de cut-off - Οριακός ορός Αντιδραστήριο βαθμονόμησης
<b>CONJ</b>	- Coniugato - Conjugé - Konjugat - Conjugado	- Conjugate - Conjugado - Σύζευγμα
<b>MP</b>	- Micropiastro rivestita - Microplaque sensibilisée - Beschichtete Mikrotiterplatte - Microplaca revestida	- Coated microtiter plate - Microplaca sensibilizada - Επικαλυμμένη μικροπλάκα
<b>WASHB</b> <b>50x</b>	- Tampone di lavaggio - Tampon de Lavage - Waschpuffer - Solução de lavagem	- Wash buffer - Solución de lavado - Ρυθμιστικό διάλυμα πλύσης
<b>SUB</b>	- Tampone substrato - Substrat - Substratpuffer - Substrato	- Substrate buffer - Tampón sustrato - Ρυθμιστικό διάλυμα υποστρώματος
<b>STOP</b>	- Reagente bloccante - Solution d'Arrêt - Stopreagenz - Solução de paragem	- Stop solution - Solución de parada - Αντιδραστήριο διακοπής αντίδρασης
<b>SB</b> <b>5x</b>	- Tampone campione - Tampon Echantillons - Probenpuffer - Diluente de amostra	- Sample buffer - Tampón Muestras - Ρυθμιστικό διάλυμα δειγμάτων